



OVAL WINDOW AUDIO "LOOPING THE WORLD"

33 Wildflower Court, Nederland, CO 80466 USA (303) 447-3607 VOICE/FAX
www.ovalwindowaudio.com • info@ovalwindowaudio.com

Portable InfoLoop Instructions

Your Portable InfoLoop is a self-contained rechargeable induction loop hearing assistance system that transmits a short-range signal to hearing aids that have the telephone ("telecoil") feature. Battery powered loop receivers may also be used.

1. Unpack and identify the following Portable InfoLoop system components:

- ___ Main unit (built in amplifier, loop antenna, microphone and rechargeable battery)
- ___ External plug in microphone: _____
- ___ AC adapter/battery charger
- ___ Accessories: _____

2. Charge the system for 24 hours before using.

3. The main unit may be placed on a desk, counter top or other location that is within three to five feet of the listener whose hearing aids have been switched to the *telecoil* mode. It should be positioned so that the front of the unit with the large decal is facing the hearing aid user, while the rear of the unit is facing the speaker/sound source. The *very best* sound is achieved by utilizing the supplied external lapel or table top microphone plugged in to the EXT jack at the rear of the unit. The microphone should be placed as close as possible to the sound source(s).

4. Turn on the system by sliding the OFF/ON switch to ON, to OFF when not in use.

5. During normal operation when the system detects sound, the green LED will flash.

6. The Portable InfoLoop will operate for 5 to 8 hours on a full charge. If the battery level is low and charging is necessary, the LED will flash slowly. The system may be used while it is charging, and may be left on charge when not in use.

Quality Control: _____ Date: _____